

How to use the epi-pen auto-injector

HOW TO USE THE EpiPen® AUTO-INJECTOR

EpiPen Jr - for children 33 - 66 lbs
EpiPen - for adults and children over 66 lbs



TWO SIMPLE STEPS

1) **Blue to the Sky**

- Grasp with orange tip pointing downward
- Remove blue safety cap by pulling straight up – do not bend or twist



2) **Orange to the thigh**

- Place the orange tip against the middle of the outer thigh
- Swing and push the auto-injector firmly into the thigh until it “clicks”
- Hold in place for 3 full seconds

Built-in needle protection

After injection, the orange cover automatically extends to ensure the needle is never exposed.

After using EpiPen®, you must seek immediate medical attention or go to the emergency room. For the next 48 hours, you must stay close to a healthcare facility or be able to call 911.

For more information visit EpiPen.ca

Adopted: November 2021